


Masimo SafetyNet Alert™



Oxygen Monitoring and Alert System

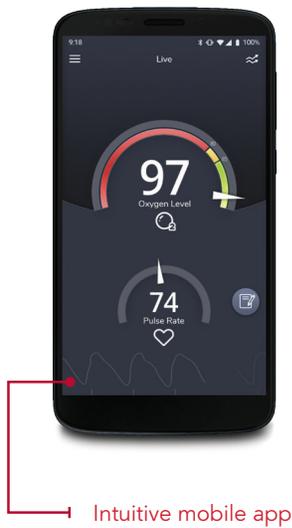

Intuitive mobile app



Home Medical Hub with audible alarms



Comfortable, disposable sensor

Components:



Masimo SafetyNet Alert App

- Displays oxygen, pulse rate, and other data
- Cloud sends alerts to the Home Medical Hub and mobile device
- Compatible with most mobile devices (mobile device not included)



Masimo Chip

- Reusable chip snaps into the sensor and communicates with the Home Medical Hub



Home Medical Hub

- Collects data from Masimo Sensor
- Sends data to smartphone app



Masimo Sensor

- 2 easy-to-apply, disposable sensors measure oxygen level and pulse rate (each single-use sensor lasts up to 96 hours in typical continuous usage*)



Replacement Tapes

- 12 replacement tapes are provided for sensor application

Compatibility Requirements:

Requirement	Specification
Wireless Connectivity	<ul style="list-style-type: none"> ▪ The Home Medical Hub is compatible with 2.4 GHz wireless networks.
Mobile Device or Smart Device Operating System	<ul style="list-style-type: none"> ▪ Android 6.0 (Marshmallow) and higher <ul style="list-style-type: none"> • Display Size 4.7" and higher ▪ iOS 13 and higher <ul style="list-style-type: none"> • Display size 4.7" and higher

Escalating Alerts Help Keep Your Patients Safe



When a patient's physiological data indicates there may be a problem, the system alerts them on the app and at the Home Medical Hub, triggering critical early opportunities to self-recover or get help.

If the patient's condition persists or gets worse, the system alerts the patient and sends automatic texts to designated emergency contacts.[†]

Users can select from three options (Opioid, COVID-19, or Other).
The selected option helps to identify the most relevant Alert Events for the user.

Opioid: Alerts are preset and are not customizable.

Alert Type	Low SpO2 Limit % Default	Low SpO2 Limit % User Settings	Time Delay Default	Time Delay User Settings
Level 1 - Caution	85%	NA	30 seconds	NA
Level 2 - Warning	80%	NA	30 seconds	NA
Level 3 - Emergency	80%	NA	180 seconds	NA
	60%	NA	60 seconds	NA

Other: Alerts have default settings, but are customizable.

Alert Type	Low SpO2 Limit % Default	Low SpO2 Limit % User Settings	Time Delay Default	Time Delay User Settings
			OTHER	
Level 1 - Caution	93%	Max: 95% Min: Warning Limit + 1%	5 minutes	30 seconds to 5 minutes in 30 second increments
Level 2 - Warning	90%	Max: Caution Limit - 1% Min: Emergency Limit + 1%	2 minutes	30 seconds to 5 minutes in 30 second increments
Level 3 - Emergency	80%	Max: Warning Limit - 1% Min: 80%	30 seconds	30 seconds to 2 minutes in 30 second increments

* Based on 96 hours of sensor battery life (not rechargeable). Remove chip between uses.

[†] Emergency contacts must agree to receive alerts.

iOS is a trademark or registered trademark of Cisco in the U.S. and other countries and is used under license. ANDROID™ is a registered trademark of Google Inc. Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG Inc.

 Masimo SafetyNet Alert has obtained CE Marking.

For professional use. See instructions for use for full prescribing information including indications, contraindications, warnings and precautions. Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

61524/PLM-13855A-0322 | © 2022 Masimo. All rights reserved.

Masimo U.S.
Tel: 1 877 4 Masimo
info-america@masimo.com

Masimo International
tel +41-32-720-1111
info-international@masimo.com

